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
COMPATIBILITY

Your U-2000 Amplifier will work with most telephones that have modular cords connecting the handset to the base. It will not work with a handset having a built-in dial. This product is hearing-aid compatible when used with a hearing-aid compatible telephone. **FOR INDOOR USE ONLY.**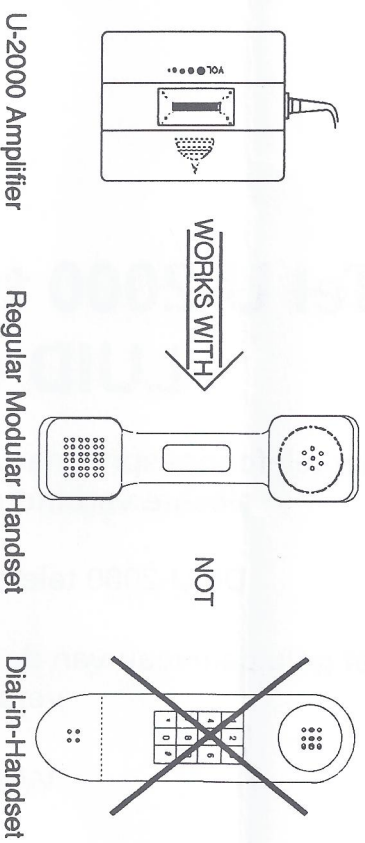

STEP 1 INSTALLING THE BATTERY

NOTE - Always unplug the Amplifier from the telephone base before opening the battery door.

- Remove the battery compartment door by gently pressing down and sliding back the battery door (Figure 1).

- Insert a standard 1.5 Volt AA battery (not included) into the battery compartment (Figure 2).

NOTE - Be sure the battery polarity is correct before closing. The amplifier will not receive battery power if the battery is installed backwards.


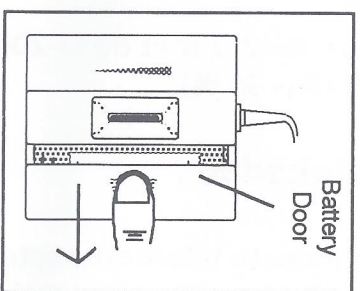
Figure 1

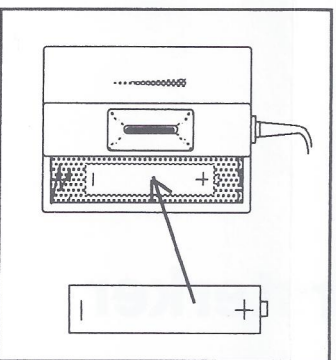
- Replace the battery compartment door by gently sliding it back into place until it "clicks" closed. Do not force it into place.

STEP 2 CONNECTING THE AMPLIFIER

- Unplug the handset cord from the telephone base and plug it into the open Amplifier jack (Figure 3). Be sure it "clicks" into place.

- Plug the Amplifier cord into the base jack where you just unplugged the handset cord. Be sure it "clicks" into place.

Figure 3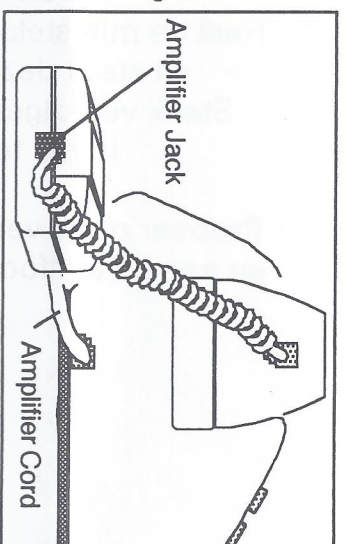

STEP 3 MOUNTING THE AMPLIFIER

If you wish, you can secure the Amplifier to a desk, a wall, or to the side of the telephone by using the included strips of SCOTCH-MATE® FASTENERS.

- Peel off the protective paper covering on each strip. Mount the sticky side of one strip to the bottom-center of the Amplifier. Do not cover the FCC label with the strip.

- Mount the sticky side of the remaining strip to the surface where you wish to attach the amplifier.

- Mount the Amplifier by securely pressing the strips together.

NOTE: The strips may leave marks on some surfaces.

ADJUSTING LISTENING LEVELS

During a call, rotate the volume control (Figure 4) to adjust incoming speech for a clear, comfortable listening level.

NOTE - At high volume settings, it is normal to hear "singing" caused by acoustic feedback if the handset is placed face down on a hard surface. This is an indication of high amplifying power.

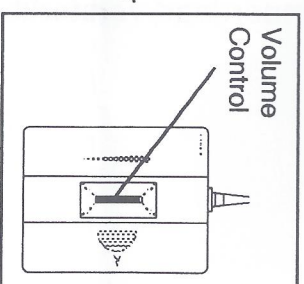
Figure 4

LOW-BATTERY INDICATION

The battery should be replaced when the handset no longer gives powerful amplification at high volume settings. This may occur when the battery is a year or so old. It is best to replace the battery about once a year (see STEP 1 INSTALLING THE BATTERY).